## Lilaset - Kadınlar İçin İdrar Drenaj Seti

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**Telefon:** +90 850 850 7570

**E-posta:** [info@lilaset.com](mailto:info@lilaset.com)

### Lilaset – Urine Drainage Set for Women

#### Instructions for Use:

Thank you for purchasing this product. Lilaset will make things easier for you or those who help you out with your urination needs. Designed as a urination drainage set for women, it is a great convenience for both bedbound women or women who need to urinate but cannot find a toilet at a particular time. Specifically designed so that it can be used under any circumstances, Lilaset will make the occasional toilet challenge for women a thing of the past.

#### Contents:

1. Lilaset Presentation
2. Lilaset Parts
3. Lilaset Assembly
4. Lilaset Use
5. Lilaset Cleaning and Storage
6. Lilaset Warnings and Useful Information

### 1. Lilaset Presentation

Lilaset was designed with women's toilet needs in mind. The set consists of two parts, the drainage appliance and the urination bag. Lilaset introduces an easy solution to your toilet needs at home, on the move, inside a vehicle, while camping and in places which are not hygienically safe, regardless whether you are lying, sitting, or standing. Lilaset is constructed of skin-friendly TPE material.

### 2. Lilaset Parts

#### Picture 1

### 3. Lilaset Assemblı

#### Picture 2

You can watch the animated Lilaset assembly video at [www.lilaset.com](http://www.lilaset.com). You can also access more information about Lilaset by scanning the QR code on the box.

### 4. Lilaset Use

#### Picture 3

- The product can be used in sitting position inside a vehicle. Illustrated Picture 4 is the vehicle sitting position for Lilaset use.
  - It can be used by women in a wheelchair or women unable to get up from their seats. Illustrated Picture 5 are the two Lilaset chair sitting positions.
  - Lilaset can be used in a standing position in open air places where there are no toilets or when the available toilets are not hygienic. Illustrated Picture 6 are the relevant standing and lying positions.
- Important Reminder: Except when you are using Lilaset in the standing position, make sure you take precautions against dripping by placing a paper towel etc. under the appliance in all sitting or lying positions.

### 5. Lilaset Cleaning and Storage

The urine drainage appliance is suitable for repeated use on condition it is washed with plenty of water and soap or detergent after each application.

After being washed, the product must definitely be dried and kept in a hygienic environment.

This product is for personal use only. It is not intended for multiple users.

### 6. Lilaset Warnings and Useful Information

- Lilaset can be used by women on their own. If you cannot use the product by yourself, people close to you can help you use it. Women who need to urinate while sitting or lying must be awake and conscious while using the product.
- The urine bag is for single use only. The urine can be drained by carefully pulling the valve under the bag.
- Do not turn the urine bag upside down because the liquid inside would flow back.
- If desired, you can use the Lilaset apparatus with other urine bags. You can cut the hose that comes with lilaset from a point close to the bag and connect the connectors of other urine bags to the end of the hose.
- Do not subject the drainage hose to pressure in lying position and make sure it is located below your knee. The drainage hose

and the urine bag should be below the level of the appliance at all times.

- Take care to ensure that a paper towel or similar material is placed under the genital area to prevent contamination by residual urine after each use. In this way, precautions will be taken against possible dripping while removing the appliance from the genital area.

- Lilaset may not function sufficiently for anatomical reasons in cases where it is used by young girls and/or excessively obese women.

- The product should be kept in a cool and dry place in room temperature without exposure to the sun.

- Care should be taken to ensure that undue pressure on the product that would change its shape is not applied.

- The life of the product can vary depending on frequency of use and storage conditions after first use. The recommended life of the product is two years subject to compliance with the storage conditions. It is suggested that the product be replaced with a new one every three months if it is used continuously or every day.

- The product is not intended for treatment purposes.

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Medhibit Medikal Sağlık Bilişim Limited